



Statement of Disinfection and Decontamination.

Incoming Materials

It is the policy and practice of Kingsbridge Dental Laboratory to inspect all incoming materials from a dental surgery, and to clean and disinfect as appropriate. The material is treated in accordance with the guidelines below:-

Impressions.

All impressions are disinfected by immersion in Disifin. Immersion is for 10 minutes in a solution mixed according to the manufactures instruction for this use. After immersion, the impressions are rinsed and cleaned under running water.

Denture try ins/bite registrations and associated models.

All dentures returned after having been tried in the patients mouth are disinfected with Disifin. Bite registrations and other items of diagnostic use that have been in contact with the oral cavity are also treated with Disifin. Any model that the above items have had probable contact with are also treated with Disifin. The Disifin for all these items is used from a spray bottle with Disifin mixed to a solution recommended for this use.

Re-Work

All jobs returned for reworking or adjustment are disinfected with Disifin from a spray bottle with the Disifin mixed to a solution recommended for this use.

Outgoing Materials

All crowns, bridges, prosthetics, temporaries and other finished items are cleaned thoroughly before being packed and dispatched. Initial cleaning is in a solution of Sodium Hydroxide. The work is then rinsed and cleaned with water before final cleaning with a steam jet at 3-4 bar pressure.

**All outgoing materials are supplied non-sterile.
MDHA Registration Number - CA2711**